




# Probe & Surface Disinfectant Wipes



## Product Usage Guide

Professional

### BA120

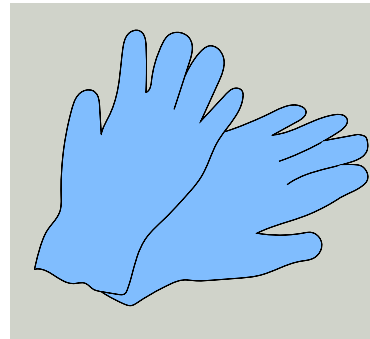

**1.** Please Use appropriate PPE as indicated in COSHH sheet.




**2.** Check pack for use period and dispense a wipe.




**3.** Wipe the surface in an S-Shape moving from clean to dirty. Use the wipe flat not scrunched.



**4.** Do not go over the same area twice with the same wipe.

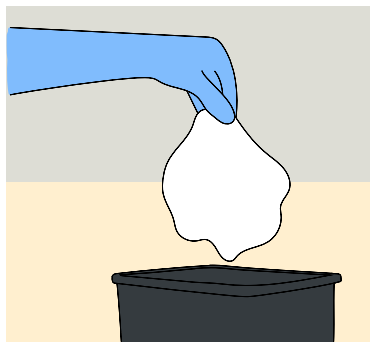

**5.** Discard used wipes in the appropriate waste bin following your local agreed guidelines.

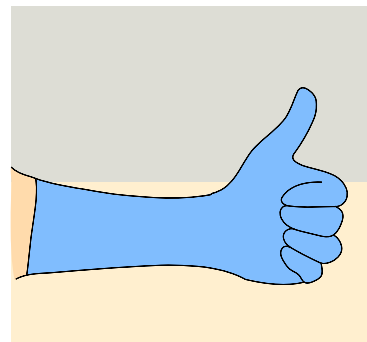

**6.** Allow the surface to dry before use.



**7.** Wash hands thoroughly after use.

### Health & Safety

Please refer to relevant COSHH Safety Data Sheet



Scan for further product details